

Reporting Title: Beta-CrossLaps (B-CTX), S
Performing Location: Rochester

- Specimen Requirements:**
Patient Preparation: Patient should fast for 12 hours before specimen collection.
Supplies: Sarstedt Aliquot Tube, 5 mL (T914)
Collection Container/Tube:
Preferred: Red top
Acceptable: Serum gel
Submission Container/Tube: Plastic vial
Specimen Volume: 1 mL
Collection Instructions:
1. Collect specimen prior to 10 a.m.
2. Centrifuge and aliquot serum into plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	90 days	
	Refrigerated	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
CTX	Beta-CrossLaps (B-CTX), S	Numeric	pg/mL	41171-0

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

CPT Code Information:
82523

Reference Values:
Males
<5 years: 242-1292 pg/mL
5-9 years: 351-1532 pg/mL
10-15 years: 447-2457 pg/mL
16-17 years: 478-1666 pg/mL
18-29 years: 238-1019 pg/mL
30-39 years: 225-936 pg/mL
40-49 years: 182-801 pg/mL
50-59 years: 161-737 pg/mL
60-69 years: 132-752 pg/mL
> or =70 years: 118-776 pg/mL

Females

<5 years: 347-1508 pg/mL

5-9 years: 383-1556 pg/mL

10-15 years: 311-1776 pg/mL

16-17 years: 146-1266 pg/mL

18-29 years: 148-967 pg/mL

30-39 years: 150-635 pg/mL

40-49 years: 131-670 pg/mL

50-59 years: 183-1060 pg/mL

60-69 years: 171-970 pg/mL

> or =70 years: 152-858 pg/mL

Premenopausal: 136-689 pg/mL

Postmenopausal: 177-1015 pg/mL