

## **Test Definition: AMS**

Amylase, Total, Serum

**Reporting Title:** Amylase, Total, S **Performing Location:** Rochester

**Necessary Information:** 

Patient's age and sex are required.

**Specimen Requirements:** 

**Collection Container/Tube:** 

**Preferred:** Serum gel **Acceptable:** Red top

**Submission Container/Tube: Plastic vial** 

**Specimen Volume:** 0.5 mL **Collection Instructions:** 

- 1. Serum gel tubes should be centrifuged within 2 hours of collection.
- 2. Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

#### Forms:

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	30 days	
	Refrigerated	30 days	
	Ambient	7 days	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
AMS	Amylase, Total, S	Numeric	U/L	1798-8

LOINC® and CPT codes are provided by the performing laboratory.

### **Supplemental Report:**

No

#### **CPT Code Information:**

82150

#### **Reference Values:**

0-30 days: 0-6 U/L 31-182 days: 1-17 U/L 183-365 days: 6-44 U/L 1-3 years: 8-79 U/L 4-17 years: 21-110 U/L > or =18 years: 28-100 U/L



# **Test Definition: AMS**

Amylase, Total, Serum