

Prolactin, Serum

**Reporting Title:** Prolactin, S **Performing Location:** Rochester

## **Specimen Requirements:**

Patient Preparation: For 12 hours before specimen collection do not take multivitamins or dietary supplements containing biotin (vitamin B7), which is commonly found in hair, skin, and nail supplements and multivitamins.
Container/Tube:
Preferred: Serum gel
Acceptable: Red top
Submission Container/Tube: Plastic vial
Specimen Volume: 0.6 mL
Collection Instructions: Centrifuge and aliquot serum into a plastic vial.

#### Forms:

If not ordering electronically, complete, print, and send an <u>Oncology Test Request</u> (T729) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	7 days	
	Frozen	90 days	
	Ambient	24 hours	

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
PRL	Prolactin, S	Numeric	ng/mL	20568-2

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No

CPT Code Information:

84146

## **Reference Values:**

Males <18 years: not established > or =18 years: 4.0-15.2 ng/mL

Females: <18 years: not established > or =18 years: 4.8-23.3 ng/mL

For SI unit Reference Values, see https://www.mayocliniclabs.com/order-tests/si-unit-conversion.html



# **Test Definition: PRL**

Prolactin, Serum