

## **Test Definition: PBU**

Lead, 24 Hour, Urine

Reporting Title: Lead, 24 Hr, U Performing Location: Rochester

#### **Ordering Guidance:**

The Centers for Disease Control and Prevention recommends venous blood collection for lead testing; see PBDV / Lead, Venous, with Demographics, Blood

#### **Necessary Information:**

24-Hour volume (in milliliters) is required.

#### **Specimen Requirements:**

**Patient Preparation:** High concentrations of gadolinium and iodine are known to interfere with most metal tests. If either gadolinium- or iodine-containing contrast media has been administered, a specimen should not be collected for 96 hours.

Supplies: Urine Tubes, 10 mL (T068)

Collection Container/Tube: Clean, plastic urine container with no metal cap or glued insert

Submission Container/Tube: Plastic, 10-mL urine tube or a clean, plastic aliquot container with no metal cap or glued

insert

Specimen Volume: 10 mL
Collection Instructions:

- 1. Collect urine for 24 hours.
- 2. Refrigerate specimen within 4 hours of completion of 24-hour collection.
- 3. See Metals Analysis Specimen Collection and Transport for complete instructions.

**Additional Information:** See <u>Urine Preservatives-Collection and Transportation for 24-Hour Urine Specimens</u> for multiple collections.

### **Urine Preservative Collection Options:**

**Note:** The addition of preservative or application of temperature controls **must occur within 4 hours of completion** of the collection.

Ambient	ОК
Refrigerate	Preferred
Frozen	ОК
50% Acetic Acid	ОК
Boric Acid	No
Diazolidinyl Urea	No
6M Hydrochloric Acid	ОК
6M Nitric Acid	ОК
Sodium Carbonate	No
Thymol	No
Toluene	No

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	28 days	
	Ambient	28 days	
	Frozen	28 days	



# **Test Definition: PBU**

Lead, 24 Hour, Urine

## Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
PBU	TM83	Collection Duration	Plain Text	Yes
PBU	VL84	Urine Volume	Plain Text	Yes

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
31085	Lead, 24 Hr, U	Numeric	mcg/24 h	5677-0
TM83	Collection Duration	Alphanumeric		13362-9
VL84	Urine Volume	Alphanumeric		3167-4

 $\ensuremath{\mathsf{LOINC}}\xspace^{\ensuremath{\texttt{@}}}$  and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

No

## **CPT Code Information:**

83655

## **Reference Values:**

0-17 years: Not established > or =18 years: <2 mcg/24 h