

Test Definition: THEO

Theophylline, Serum

Reporting Title: Theophylline, S
Performing Location: Rochester

Specimen Requirements:

Container/Tube:
Preferred: Serum gel
Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL **Collection Instructions:**

- 1. Serum gel tubes should be centrifuged within 2 hours of collection.
- 2. Red-top tubes should be centrifuged and the serum aliquoted into a plastic vial within 2 hours of collection.

Forms:

If not ordering electronically, complete, print, and send a <u>Therapeutics Test Request</u> (T831) with the specimen.

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Refrigerated (preferred) | 14 days | |
| | Frozen | 28 days | |
| | Ambient | 72 hours | |

Result Codes:

| Result ID | Reporting Name | Туре | Unit | LOINC® |
|-----------|-----------------|---------|--------|--------|
| 8661 | Theophylline, S | Numeric | mcg/mL | 4049-3 |

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80198

Reference Values:

Therapeutic:

Bronchodilation: 8.0-20.0 mcg/mL

Neonatal apnea (< or =4 weeks old): 6.0-13.0 mcg/mL

Critical value: >20.0 mcg/mL