

Test Definition: JAKXB

JAK2 Exon 12 and Other Non-V617F Mutation Detection, Blood

Reporting Title: JAK2 Exon 12 Mutation Detection, B **Performing Location:** Rochester

Ordering Guidance:

In all cases being evaluated for *JAK2* mutation status, the initial test that should be ordered is JAK2B / *JAK2* V617F Mutation Detection, Blood, a sensitive assay for detection of the mutation. However, if no *JAK2* V617F mutation is found, further evaluation of *JAK2* may be clinically indicated.

Additional Testing Requirements:

Shipping Instructions:

Specimen must arrive within 5 days (120 hours) of collection. Collect and package specimen as close to shipping time as possible.

Necessary Information: Date of collection is required.

Specimen Requirements:

Container/Tube: Preferred: Lavender top (EDTA) Acceptable: Yellow top (ACD) Specimen Volume: 10 mL

Collection Instructions:

1. Invert several times to mix blood.

2. Send whole blood specimen in original tube. Do not aliquot.

Forms:

1. <u>Hematopathology Patient Information</u> (T676)

2. If not ordering electronically, complete, print, and send a <u>Hematopathology/Cytogenetics Test Request</u> (T726) with the specimen.

Specimen Type	Temperature	Time	Special Container	
Whole blood	Refrigerated (preferred)	5 days	PURPLE OR PINK TOP/EDTA	
	Ambient	5 days	PURPLE OR PINK TOP/EDTA	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
20194	Final Diagnosis:	Alphanumeric		34574-4
39467	JAK2 Sequencing Result	Alphanumeric		55300-8

LOINC[®] and CPT codes are provided by the performing laboratory.



Test Definition: JAKXB

JAK2 Exon 12 and Other Non-V617F Mutation Detection, Blood

Supplemental Report:

No

CPT Code Information:

0027U-JAK2 (Janus kinase 2) (eg, myeloproliferative disorder), exon 12 sequence and exon 13 sequence, if performed

Reference Values:

An interpretive report will be provided.