

Reporting Title: 6-MAM Confirmation, U
Performing Location: Rochester

Ordering Guidance:

- 1. For situations where chain of custody is required, a Chain-of-Custody Kit (T282) is available. For chain-of-custody testing, order 6MAMX / 6-Monoacetylmorphine, Chain of Custody, Random, Urine.
- 2. If urine creatinine is required or adulteration of the sample is suspected, order ADULT / Adulterants Survey, Random, Urine.
- 3. Additional drug panels and specific requests are available. Call 800-533-1710 or 507-266-5700.

Specimen Requirements:

Supplies: Urine Tubes, 10 mL (T068)
Container Tube: Plastic, 10-mL urine tube
Specimen Volume: 10 mL

Collection Instructions:

- 1. Collect a random urine specimen.
- 2. No preservative.

Additional Information:

- 1. No specimen substitutions.
- 2. STAT requests are **not** accepted for this test.

Forms:

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	14 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
30983	6-Monoacetylmorphine by GC/MS	Alphanumeric	ng/mL	19593-3
29898	6-MAM Interpretation	Alphanumeric		69050-3
29899	Chain of Custody	Alphanumeric		77202-0

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80356
G0480 (if appropriate)

Reference Values:
Negative

6-MAM cutoff concentration: <5 ng/mL