

Reporting Title: Fluticasone 17-B Carboxylic Acid, U
Performing Location: Rochester

Specimen Requirements:
Collection Container/Tube: Clean, plastic urine collection container
Submission Container/Tube: Plastic, 10-mL urine tube (T068)
Specimen Volume: 5 mL
Collection Instructions:
1. Collect a random urine specimen.
2. No preservative.

Specimen Type	Temperature	Time	Special Container
Urine	Frozen (preferred)	14 days	
	Refrigerated	72 hours	
	Ambient	24 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
89739	Fluticasone 17-B Carboxylic Acid, U	Numeric	pg/mL	46952-8

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

CPT Code Information:
80299

Reference Values:
Negative
Cutoff concentration: 10 pg/mL

Values for normal patients not taking fluticasone propionate should be less than the cutoff concentration (detection limit).