

# **Test Definition: FOXAZ**

Oxazepam (Serax), Serum

Reporting Title: Oxazepam (Serax)

Performing Location: Medtox Laboratories, Inc.

## **Specimen Requirements:**

Submit only 1 of the following specimens:

#### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z1141	Oxazepam (Serax)	Alphanumeric		59730-2

LOINC® and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

#### **CPT Code Information:**

80346

G0480 (if appropriate)

# **Reference Values:**

Reference Range: 200 - 500 ng/mL