

Osmotic Fragility, Erythrocytes

# **Reporting Title:** Osmotic Fragility, RBC **Performing Location:** Rochester

Ordering Guidance:

**Additional Testing Requirements:** 

### Shipping Instructions:

Specimens must arrive within 72 hours of collection.

## Necessary Information:

Patient's age is required.

### Specimen Requirements:

**Both a whole blood EDTA specimen and a shipping control specimen are required.** The shipping control specimen is used to evaluate whether a patient result has been compromised by handling conditions such as temperature, motion, or other transportation interferences. Temperature and handling extremes can adversely impact the integrity of the specimen.

#### Patient:

Specimen Type: Whole blood
Container/Tube: Lavender top (EDTA)
Specimen Volume: 4 mL
Collection Instructions:

Refrigerate specimen immediately after collection.
Send whole blood specimen in original tube. Do not aliquot.

3. Rubber band patient specimen and control vial together.

### Normal Shipping Control:

Specimen Type: Whole blood Container/Tube: Lavender top (EDTA)

container/Tube. Lavender top (EDTA

# Specimen Volume: 4 mL

Collection Instructions:

1. Collect a shipping control specimen from a normal (healthy), unrelated, nonsmoking person at the same time as the patient.

- 2. Clearly hand write "normal control" on the outermost label.
- 3. Refrigerate specimen immediately after collection.
- 4. Send control specimen in original tube. Do not aliquot.

5. Rubber band patient specimen and control vial together. The control and patient specimen must be handled in the same manner from specimen collection to receipt in the testing laboratory.

### Forms:

If not ordering electronically, complete, print, and send a <u>Benign Hematology Test Request</u> (T755) with the specimen.

**Test Definition: FRAG** 

Osmotic Fragility, Erythrocytes

Specimen Type	Temperature	Time	Special Container
Control	Refrigerated	72 hours	PURPLE OR PINK TOP/EDTA
Whole Blood EDTA	Refrigerated	72 hours	

### Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
9064	Osmotic Fragility, RBC	Numeric		34964-7
3306	Osmotic Fragility, 0.50 g/dL NaCl	Numeric	%hemol	23915-2
3307	Osmotic Fragility, 0.60 g/dL NaCl	Numeric	%hemol	23918-6
3308	Osmotic Fragility, 0.65 g/dL NaCl	Numeric	%hemol	23920-2
3309	Osmotic Fragility, 0.75 g/dL NaCl	Numeric	%hemol	23921-0
3310	Osmotic Fragility Comment	Alphanumeric		59466-3
SCTRL	Shipping Control Vial	Alphanumeric		40431-9

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

MAYO CLINIC LABORATORIES

#### Supplemental Report:

No

#### **Components:**

Test Id	Reporting Name	CPT Units	CPT Code	Always Performed	Available Separately
FRAGO	Osmotic Fragility	1	85557	Yes	No
SCTRL	Shipping Control Vial			Yes	No

## **CPT Code Information:**

85557

### **Reference Values:**

> or =12 months:
0.50 g/dL NaCl (unincubated): 3-53% hemolysis
0.60 g/dL NaCl (incubated): 14-74% hemolysis
0.65 g/dL NaCl (incubated): 4-40% hemolysis
0.75 g/dL NaCl (incubated): 1-11% hemolysis

NaCl = sodium chloride Reference values have not been established for patients who are younger than 12 months of age.