

Reporting Title: Glipizide (Glucotrol)  
Performing Location: Medtox Laboratories, Inc.

Specimen Requirements:  
Submit only 1 of the following specimens:

**Plasma**  
Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable**. Spin down and send 2 mL sodium heparin plasma refrigerated in a plastic vial.

**Serum**  
Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z1180	Glipizide	Alphanumeric		10539-5

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:  
No

CPT Code Information:  
80299

Reference Values:  
Units:   ng/mL

Plasma insulin concentrations have been shown to increase only when plasma glipizide concentrations exceeded 200 ng/mL.

Toxic range has not been established.