

# Reporting Title: Furosemide (Lasix)

Performing Location: Medtox Laboratories, Inc.

# **Specimen Requirements:**

### Submit only 1 of the following specimens:

### Plasma

Draw blood in a green-top (sodium heparin) tube(s), **plasma gel tube is not acceptable.** Spin down and send 1 mL sodium heparin plasma refrigerated in a plastic vial.

#### Serum

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable.** Spin down and send 1 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container	
Varies	Refrigerated (preferred)	7 days		
	Frozen	180 days		
	Ambient	72 hours		

### Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
Z1189	Furosemide	Alphanumeric		3659-0

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

### Supplemental Report:

No

# **CPT Code Information:**

80299

# **Reference Values:**

Expected serum furosemide concentration in patients on usual daily dosages: Up to 5.0 ug/mL

Toxic: greater than 50.0 ug/mL