

Reporting Title: Anti-Phosphatidylethanolamine Panel

Performing Location: BioAgilytix Diagnostics

Specimen Requirements:

Specimen Type: Serum

Container/Tube: Red Top or SST

Specimen Volume: 3 mL

Collection Instructions: Draw blood in a plain ed-top tube(s), serum gel tube is acceptable. After collection, allow blood to clot for 30 minutes. Spin down and send 3 mL of serum frozen in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	30 days	
	Refrigerated	14 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
Z0150	Anti-Phosphatidylethanolamine IgG	Numeric	U/mL	13076-5
Z0142	Anti-Phosphatidylethanolamine IgM	Numeric	U/mL	13077-3
Z0143	Anti-Phosphatidylethanolamine IgA	Numeric	U/mL	13078-1

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83520 x 3

Reference Values:

Anti-Phosphatidylethanolamine IgA

<12.0 U/mL

Anti-Phosphatidylethanolamine IgG

<12.0 U/mL

Anti-Phosphatidylethanolamine IgM

<12.0 u/mL

Reference Range applies to Anti-Phosphatidylethanolamine IgA, IgG, & IgM

Normal: <12.0

Equivocal: 12.0-18.0
Elevated: >18.0