

# Reporting Title: Isoniazid (INH), S

Performing Location: Medtox Laboratories, Inc.

## Specimen Requirements:

Specimen Type: Serum

Container/Tube: Red-top

Specimen Volume: 2 mL

**Collection Instructions**: Draw blood in a plain red-top tube(s), serum gel tube is not acceptable. Spin down and send 2 mL of serum refrigerated in a plastic vial.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
Z1207	Isoniazid (INH), S	Alphanumeric		3697-0

LOINC<sup>®</sup> and CPT codes are provided by the performing laboratory.

#### Supplemental Report:

No

#### **CPT Code Information:**

80299

## **Reference Values:**

Units: ug/mL

The effective concentration range of isoniazid is dependent upon the minimum inhibitory concentration of the pathogen being treated.

Toxic range: greater than 20 ug/mL