

**Reporting Title:** Avian Panel  
**Performing Location:** Medical College of WI Allergy-Immunology MACC Fund Research Ctr.

**Specimen Requirements:**  
**Collection Container/Tube:** Red Top or SST  
**Submission Container/Tube:** Plastic vial  
**Specimen Volume:** 3 mL  
**Collection Instructions:**

- 1. Draw blood in a Red-top tube(s). Serum-gel tube(s) is also acceptable.
- 2. Centrifuge and send 3 mL of serum refrigerated.

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	30 days	
	Frozen	365 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
Z0289	Pigeon DE	Alphanumeric		Not Provided
Z0285	Parakeet	Alphanumeric		Not Provided
Z0286	Cockatiel	Alphanumeric		Not Provided
Z0287	Parrot	Alphanumeric		Not Provided
Z0288	Pigeon Sera	Alphanumeric		Not Provided

LOINC® and CPT codes are provided by the performing laboratory.

**Supplemental Report:**  
No

**CPT Code Information:**  
86331x5

**Reference Values:**  
This panel includes the following antigens:

- Pigeon Sera
- Pigeon DE
- Cockatiel
- Parakeet
- Parrot

Results are reported as:  
Negative

Weakly Positive

Positive

Strong Positive

This result must be correlated with patient's clinical response and should not solely be considered in the diagnosis.