

Reporting Title: Cocaine and metabolite Conf, U
Performing Location: Rochester

Ordering Guidance:

- 1. For situations where chain of custody is required, a Chain-of-Custody Kit (T282) is available. For chain-of-custody testing, order COKEX / Cocaine and Metabolite Confirmation, Chain of Custody, Random, Urine.
- 2. Additional drug panels and specific requests are available. Call 800-533-1710 or 507-266-5700.
- 3. If urine creatinine is required or adulteration of the sample is suspected, order ADULT / Adulterants Survey, Random, Urine.

Specimen Requirements:

- Supplies: Urine Tubes, 10 mL (T068)
- Collection Container/Tube: Plastic urine container
- Submission Container/Tube: 10-mL urine tube
- Specimen Volume: 10 mL
- Collection Instructions:
- 1. Collect a random urine specimen.
 - 2. No preservative.
- Additional Information:
- 1. No specimen substitutions.
 - 2. STAT requests are **not** accepted for this test.

Forms:

If not ordering electronically, complete, print, and send a [Therapeutics Test Request](#) (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	14 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
2922	Cocaine-by GC/MS	Alphanumeric	ng/mL	20519-5
2902	Benzoyllecgonine-by GC/MS	Alphanumeric	ng/mL	16226-3
20671	Cocaine Interpretation	Alphanumeric		69050-3
28080	Chain of Custody	Alphanumeric		77202-0

LOINC® and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80353
G0480 (if appropriate)

Reference Values:

Negative
Positive results are reported with a quantitative gas chromatography mass spectrometry (GC-MS) result.
Cutoff concentrations:
COCAINE BY GC-MS
<50 ng/mL

BENZOYLECGONINE BY GC-MS
<50 ng/mL