

# **Test Definition: FARWB**

Anti-retinal autoantibodies follow up, WB

## Overview

## **Special Instructions**

• OHSU Requisition Form

#### **Method Name**

Western Blot (WB)

## **NY State Available**

Yes

# Specimen

## Specimen Type

Varies

#### Specimen Required

**Note:** This test should only be ordered as follow up to previous Anti-retinal autoantibodies performed at Oregon Health Sciences University (OHSU) Ocular Immunology Laboratory.

## Submit only one of the following specimens:

# Serum:

Draw blood in a plain, red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 5 mL serum refrigerated in a plastic vial.

#### Plasma:

Draw blood in a lavender-top (EDTA) tube(s). Spin down and send 5 mL EDTA plasma refrigerated in a plastic vial.

Complete and submit with specimen:

- 1. Completed OHSU Ocular request form
- 2. Clinical history
- 3. Referring physician information (name & phone number)

-NOTE: Without this information, testing cannot be completed.

## Specimen Minimum Volume

3 mL

# **Reject Due To**

Hemolysis:	Mild reject; Gross reject
------------	---------------------------



# **Test Definition: FARWB**

Anti-retinal autoantibodies follow up, WB

Thawing: Warm reject; Cold OK

# **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated	7 days	

# **Clinical & Interpretive**

#### **Reference Values**

A final report will be attached in MayoAccess.

#### **Performance**

# **PDF Report**

Referral

#### Day(s) Performed

Batched

# **Report Available**

16 to 35 days

# **Performing Laboratory Location**

Ocular Immunology Laboratory OHSU

# Fees & Codes

## **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

# **CPT Code Information**

84182

# **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
---------	-----------------	--------------------



# **Test Definition: FARWB**

Anti-retinal autoantibodies follow up, WB

FARWB	Anti-Retinal Autoantibody, WB	Not Provided
Result ID	Test Result Name	Result LOINC® Value
Result ID	rest result Name	Result Louve Value
FARWB	Anti-Retinal Autoantibody, WB	Not Provided