

Anti-IgE

### Overview

Method Name

ELISA

NY State Available

Yes

### Specimen

Specimen Type Serum

#### **Specimen Required**

Specimen Type: Serum

Container/Tube: Red Top

Specimen Volume: 1.0 mL

**Collection Instructions:** Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 1.0 mL of serum ambient in a plastic vial.

#### **Specimen Minimum Volume**

0.5 mL

#### **Reject Due To**

Gross	Reject
hemolysis	
Gross lipemia	Reject
Gross icterus	Reject

#### **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Serum	Ambient (preferred)	7 days	
	Frozen	28 days	
	Refrigerated	7 days	

# **Clinical & Interpretive**

#### **Clinical Information**



This ELISA measures IgG antibodies specific for IgE. A result of normal indicates that the level of IgG anti-IgE antibodies is similar to that seen in a population of healthy individuals. A result of elevated indicates an increased level of IgG anti-IgE antibodies compared to healthy individuals. These autoantibodies have been implicated as a causative agent in autoimmune chronic urticaria and atopic dermatitis.

#### **Reference Values**

Normal

Performance

PDF Report No

Day(s) Performed

Wednesday

# **Report Available**

7 to 11 days

Performing Laboratory Location

Eurofins Viracor

# Fees & Codes

#### Fees

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact Customer Service.

#### **Test Classification**

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

#### **CPT Code Information**

83516

#### LOINC<sup>®</sup> Information

Test ID	Test Order Name	Order LOINC <sup>®</sup> Value
FANTI	Anti-IgE	Not Provided





Anti-IgE

Result ID	Test Result Name	Result LOINC <sup>®</sup> Value
FANTI	Anti-IgE	Not Provided