

Overview

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FCOKR	Cocaine, MS, WB/SP Rfx	No	No

Testing Algorithm

If the Cocaine Analysis, whole blood is positive, then the Cocaine, Confirmation (FCOKR) will be performed at an additional charge.

Method Name

Immunoassay (IA)

Gas Chromatography with mass spectrometry (GC/MS); confirmation

NY State Available

Yes

Specimen

Specimen Type

Whole blood

Specimen Required

Draw blood in a purple-top (EDTA), green-top (sodium-heparin) or grey-top (NaF/oxalate) tube(s). Send 7 mL of whole blood refrigerated in a plastic vial.

Specimen Minimum Volume

3 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
---------------	-------------	------	-------------------

Whole blood	Refrigerated (preferred)	14 days	
	Frozen	180 days	
	Ambient	72 hours	

Clinical & Interpretive

Reference Values

Cocaine Screen:  
Cocaine and Metabolite, UA – Negative; Cutoff: 25 ng/mL

Cocaine Confirmation:  
Cocaine  
Benzoyllecgonine  
Confirmation threshold: 10 ng/mL

Performance

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 9 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

CPT Code Information

80307  
80353-if applicable

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FCOKE	Cocaine, Scr w/Conf, WB	16633-0

Result ID	Test Result Name	Result LOINC® Value
Z4826	Cocaine and Metabolite	73823-7