

# **Test Definition: FURA**

Uranium, Urine

# **Overview**

## **Method Name**

Inductively Coupled Plasma-Mass Spectrometry (ICP-MS)

#### **NY State Available**

Yes

# **Specimen**

# **Specimen Type**

Urine

# **Specimen Required**

\*\*\*Must submit one specimen per order. Specimens cannot be shared between multiple orders.\*\*\*

Submission Container/Tube: Acid washed or trace metal free plastic container (T619).

Specimen Volume: 1 mL

**Collection Instructions:** Collect urine in acid washed or trace metal free plastic container (T619). Submit 1 mL of urine refrigerated.

# **Specimen Minimum Volume**

0.4 mL

## **Reject Due To**

## **Specimen Stability Information**

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	16 days	Acid Washed Plastic (MML
			Supply T619)
	Frozen	30 days	Acid Washed Plastic (MML
			Supply T619)
	Ambient	9 days	Acid Washed Plastic (MML
			Supply T619)

# **Clinical & Interpretive**

#### **Reference Values**

Reporting limit determined each analysis



# **Test Definition: FURA**

Uranium, Urine

Normally: Less than 0.1 mcg/L

## Interpretation

Specimens for elemental testing should be collected in certified metal-free containers. Elevated results for elemental testing may be caused by environmental contamination at the time of specimen collection and should be interpreted accordingly. It is recommended that unexpected elevated results be verified by testing another specimen.

#### **Performance**

# **PDF Report**

No

# Day(s) Performed

**Varies** 

# **Report Available**

9 to 18 days

#### **Specimen Retention Time**

2 weeks

## **Performing Laboratory Location**

**NMS Labs** 

## **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

## **Test Classification**

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

## **CPT Code Information**

83018

# **LOINC®** Information

Test ID	Test Order Name	Order LOINC® Value
FURA	Uranium, U	5752-1



# **Test Definition: FURA**

Uranium, Urine

Result ID	Test Result Name	Result LOINC® Value
Z1442	Uranium	5752-1
Z1874	Reporting Limit	19147-8