



# Test Definition: FWFHG

Venom W-F Hornet IgG

## Overview

### Method Name

Phadia ImmunoCAP

### NY State Available

Yes

## Specimen

### Specimen Type

Serum

### Specimen Required

#### Collection Container/Tube:

**Preferred:** Red top

**Acceptable:** Serum gel

**Submission Container/tube:** Plastic vial

**Specimen Volume:** 0.5 mL Serum

#### Collection Instructions:

1. Centrifuge and aliquot 0.5 mL of serum into a plastic vial.
2. Send refrigerated.

### Specimen Minimum Volume

See Specimen Required

### Reject Due To

## Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	28 days	
	Ambient	7 days	
	Frozen	365 days	

## Clinical & Interpretive

### Clinical Information

Refer to [www.eurofins-viracor.com/test-menu/](http://www.eurofins-viracor.com/test-menu/)

---

**Reference Values**

>3.5 mcg/mL

Venom IgG Reference Values: 1.0-3.5 mcg/mL

Low venom IgG indicating a significant reaction risk. 3.5-6.0 mcg/mL

Moderate level of venom IgG that may be associated from serious sting reactions(J Clin Immunol.1983;6:172) >6.0 mcg/mL

Elevated venom IgG usually associated with protection from serious sting reactions(J Clin Immunol.1983;6:172)

The interpretative guidelines have been adapted from Golden et al. (JACI. 1992;90:386-393).

The test method used is the Phadia ImmunoCAP IgG assay, which has been recalibrated to correlate with the venom IgG assay referenced above.

**Performance****PDF Report**

No

**Day(s) Performed**

Monday through Friday

**Report Available**

2 to 7 days

**Performing Laboratory Location**

Eurofins Viracor

**Fees & Codes****Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

**Test Classification**

[This test was developed and its performance characteristics determined by Viracor Eurofins. It has not been cleared or approved by the U.S. Food and Drug Administration.](#)

**CPT Code Information**

86001

**LOINC® Information**

---

Test ID	Test Order Name	Order LOINC® Value
FWFHG	WF Hornet IgG	49745-3

Result ID	Test Result Name	Result LOINC® Value
Z3564	Venom W-F Hornet (D-maculata) IgG	49745-3