



Test Definition: FDLS

Donath Landsteiner

Overview

Method Name

Hemolysis

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

Collection Container/Tube: Red top (serum gel/SST are **not acceptable**)

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL serum

Collection Instructions:

1. Maintain specimen at 37 degrees C until after centrifugation.
2. Centrifuge and aliquot 3 mL of serum into a plastic vial.
3. Send refrigerated.

Specimen Minimum Volume

Serum: 2 mL

Reject Due To

Gross hemolysis	Reject
Gross lipemia	Reject
Gross icterus	Reject

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	7 days	
	Ambient	72 hours	

Clinical & Interpretive

Clinical InformationRefer to <https://ltd.aruplab.com/>**Reference Values**

Negative

Performance**PDF Report**

No

Day(s) Performed

Monday through Friday

Report Available

1 to 7 days

Performing Laboratory Location

ARUP Laboratories

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

86940

86941

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FDLS	Donath Landsteiner	18287-3

Result ID	Test Result Name	Result LOINC® Value
FDLS	Donath Landsteiner	18287-3