



Test Definition: FASCC

Amphetamines Analysis, Serum

Overview

Reflex Tests

Test Id	Reporting Name	Available Separately	Always Performed
FASCR	Amphetamines, Rfx	No	No

Testing Algorithm

If the Amphetamine, Scr w-Conf, S is positive, then the Amphetamines, Rfx (FASCR) will be performed at an additional charge.

Method Name

Immunoassay (IA) â€” Screen

Gas Chromatography/Mass Spectrometry (GS/MS) â€” Confirmation; if applicable

NY State Available

Yes

Specimen

Specimen Type

Serum Red

Specimen Required

Draw blood in a plain red-top tube(s), **serum gel tube is not acceptable**. Spin down and send 7 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

3 mL

Reject Due To

Hemolysis	NA
Lipemia	NA
Icterus	NA
Other	NA

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	14 days	

	Ambient	72 hours	
	Frozen	180 days	

Clinical & Interpretive

Reference Values

Reference Range:

Amphetamines: Cutoff: 50

Confirmation Threshold: 10 mg/mL

Performance

PDF Report

No

Day(s) Performed

Monday through Sunday

Report Available

5 to 9 days

Performing Laboratory Location

Medtox Laboratories, Inc.

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

CPT Code Information

80307

80324 (if appropriate)

80359 (if appropriate)

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FASCC	Amphetamines, Scr w-Conf, S	8149-7

Result ID	Test Result Name	Result LOINC® Value
Z4631	Amphetamines	8149-7