

## **Test Definition: FCARO**

Carotene, Beta

## **Overview**

#### **Useful For**

Confirm the diagnosis of carotenoderma; detect fat malabsorption; depressed carotene levels may be found in cases of steatorrhea.

#### **Method Name**

High-pressure liquid chromatography (HPLC)

#### **NY State Available**

Yes

## **Specimen**

## **Specimen Type**

Serum

## **Specimen Required**

Supplies: Sarstedt 5 mL Aliquot Tube (Amber) (T915)

Specimen Type: Serum Container/Tube: Red Specimen Volume:1 mL

Collection Instructions: Draw blood in a plain, red-top tube(s). (Serum gel tube is acceptable.)

Spin Down and send 1 mL of serum ambient in an amber vial to protect from light.

## Note:

- 1. Protect from light within 1 hour of collection.
- 2. Patient must be fasting overnight (12 hours). Blood should be collected before breakfast in the morning and prior to any medication.
- 3. Abstain from alcohol for 24 hours prior to collection.

## Specimen Minimum Volume

0.5 mL

## **Reject Due To**

Hemolysis:	Mild reject; Gross reject
Thawing:	Warm OK; Cold OK
Other:	Specimen other than serum Red or SST, or if not light protected.

## **Specimen Stability Information**



# **Test Definition: FCARO**

Carotene, Beta

Specimen Type	Temperature	Time	Special Container
Serum	Ambient (preferred)	14 days	LIGHT PROTECTED
	Frozen	14 days	LIGHT PROTECTED
	Refrigerated	14 days	LIGHT PROTECTED

## **Clinical & Interpretive**

#### **Reference Values**

3 - 91 ug/dL

## Interpretation

High levels are useful to rule out steatorrhea but lower values lack specificity. There is poor sensitivity. High in the serum of those ingesting large amounts of vegetables.

#### **Performance**

#### **PDF Report**

No

#### Day(s) Performed

Monday, Wednesday, Friday

### Report Available

5 to 10 days

## **Performing Laboratory Location**

LabCorp Burlington

#### **Fees & Codes**

#### **Fees**

- Authorized users can sign in to <u>Test Prices</u> for detailed fee information.
- Clients without access to Test Prices can contact <u>Customer Service</u> 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact <u>Customer Service</u>.

#### **CPT Code Information**

82380

#### **LOINC®** Information



# **Test Definition: FCARO**

Carotene, Beta

Test ID	Test Order Name	Order LOINC® Value
FCARO	Carotene, Beta	2053-7

Result ID	Test Result Name	Result LOINC® Value
FCARO	Carotene, Beta	2053-7