



Test Definition: FPINP

Pine Ponderosa IgE

Overview

Method Name

Enzyme Immunoassay (EIA)

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Specimen Type: Serum

Container/Tube: Red or SST

Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain red-top tube(s), serum gel tube(s) is acceptable. Spin down and send 0.5 mL of serum refrigerated in a plastic vial.

Specimen Minimum Volume

0.5 mL

Reject Due To

| | |
|---------|--------|
| Lipemia | Reject |
|---------|--------|

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Serum | Refrigerated (preferred) | 28 days | |
| | Ambient | 28 days | |
| | Frozen | 365 days | |

Clinical & Interpretive

Reference Values

<0.35 kU/L

Interpretation

| Class | IgE (kU/L) | Comment |
|-----------------------|----------------------------|-------------------------|
| 0 | <0.35 | Below Detection |
| 1 | 0.35-0.69 | Low Positive |
| 2 | 0.70-3.49 | Moderate Positive |
| 3 | 3.50-17.49 | Positive |
| 4 | 17.50-49.99 | Strong Positive |
| 5 | >49.99 | Very Strong Positive |

Performance

Method Description

PDF Report

No

Day(s) Performed

Monday through Friday

Report Available

4 to 7 days

Performing Laboratory Location

Eurofins Viracor

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by Eurofins Viracor. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

86003

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-----------------|--------------------|
|---------|-----------------|--------------------|

| | | |
|-------|--------------------|--------|
| FPINP | Pine Ponderosa IgE | 7610-9 |
|-------|--------------------|--------|

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|--------------------|---------------------|
| Z5335 | Pine Ponderosa IgE | 7610-9 |
| Z5336 | CLASS | Not Provided |