



Test Definition: FCDSU

Comprehensive Drug Screen, Umbilical Cord
Tissue

Overview

Profile Information

| Test Id | Reporting Name | Available Separately | Always Performed |
|---------|-----------------------------|----------------------|------------------|
| FCDU1 | Opiates, UC | No | Yes |
| FCDU2 | Benzodiazepines, UC | No | Yes |
| FCDU3 | Acetyl Fentanyl, UC | No | Yes |
| FCDU4 | Novel Psychoactives, UC | No | Yes |
| FCDU5 | Amphetamines, UC | No | Yes |
| FCDU6 | Cocaine and Metabolites, UC | No | Yes |
| FCDU7 | Cannabinoids, UC | No | Yes |
| FCDU8 | Dextro/Levo Methorphan, UC | No | Yes |
| FCDU9 | Muscle Relaxants, UC | No | Yes |
| FCDUA | Phencyclidine, UC | No | Yes |
| FCDUB | Barbiturates, UC | No | Yes |
| FCDUE | Gabapentin, UC | No | Yes |
| FCDUD | Zolpidem, UC | No | Yes |

Reflex Tests

| Test Id | Reporting Name | Available Separately | Always Performed |
|---------|-------------------------------------|----------------------|------------------|
| FCDV1 | Acetyl Fentanyl Conf (Qual), UC | No | No |
| FCDV2 | Amphetamines Conf (Qual), UC | No | No |
| FCDV3 | Barbiturates Conf (Qual), UC | No | No |
| FCDV4 | Benzodiazepines Conf (Qual), UC | No | No |
| FCDV5 | Buprenorphine/Metab Conf (Qual), UC | No | No |
| FCDV6 | Cannabinoids Conf (Qual), UC | No | No |
| FCDV7 | Carisoprodol/Metab Conf (Qual), UC | No | No |
| FCDV8 | Cocaine/Metab Conf (Qual), UC | No | No |
| FCDV9 | Dextro/Levo Methorphan | No | No |

| | | | |
|-------|-------------------------------------|----|----|
| | Conf Qual, UC | | |
| FCDW1 | Fentanyl/Metab Conf (Qual), UC | No | No |
| FCDW2 | Meperidine/Metab Conf (Qual), UC | No | No |
| FCDW3 | Methadone/Metab Conf (Qual), UC | No | No |
| FCDW4 | Novel Psychoactive Conf (Qual), UC | No | No |
| FCDW5 | Opiates-Free Unconj Conf (Qual), UC | No | No |
| FCDW6 | Phencyclidine Conf (Qual), UC | No | No |
| FCDW7 | Tapentadol Conf (Qual), UC | No | No |
| FCDW8 | Tramadol/Metab Conf (Qual), UC | No | No |
| FCDW9 | Zolpidem Conf (Qual), UC | No | No |
| FCDX1 | Gabapentin Conf (Qual), UC | No | No |
| FCDX2 | Mitragynine Conf (Qual), UC | No | No |
| FCDX3 | Xylazine Conf (Qual), UC | No | No |

Testing Algorithm

If appropriate, confirmation testing will be performed at no additional charge.

Method Name

High Performance Liquid Chromatography/Tandem Mass Spectrometry (LC-MS/MS)

NY State Available

Yes

Specimen

Specimen Type

Tissue

Specimen Required

*****Must submit one specimen per order. Specimens cannot be shared between multiple orders.*****

Specimen Type: Umbilical cord tissue

Container/Tube: Plastic, preservative-free container

Specimen Volume: 10 g

Collection Instructions:

1. Collect at least 6 inches of umbilical cord (approximately the length of an adult hand).
2. Drain and discard any blood.
3. Rinse the exterior of the cord segment with normal saline or sterile water.
4. Pat the cord dry and place in a preservative-free plastic container for transport.
5. Ship refrigerated.

Specimen Minimum Volume

See Specimen Required

Reject Due To

| | |
|---------------------------------------|--------|
| Cords soaking in blood or other fluid | Reject |
| Formalin fixed | Reject |
| Tissue that is obviously decomposed | Reject |
| Samples received without cold pack | Reject |

Specimen Stability Information

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|---------|-------------------|
| Tissue | Refrigerated (preferred) | 7 days | |
| | Frozen | 14 days | |

Clinical & Interpretive

Clinical Information

Refer to www.nmslabs.com/test-catalog

Reference Values

Reporting limit(s) determined each analysis.

None Detected

Performance**PDF Report**

No

Day(s) Performed

Monday through Sunday

Report Available

3 to 7 days

Performing Laboratory Location

NMS Labs

Fees & Codes**Fees**

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by NMS Labs. It has not been cleared or approved by the U.S. Food and Drug Administration.

CPT Code Information

80307

LOINC® Information

| Test ID | Test Order Name | Order LOINC® Value |
|---------|-------------------------------|--------------------|
| FCDSU | Comprehensive Drug Screen, UC | Not Provided |

| Result ID | Test Result Name | Result LOINC® Value |
|-----------|------------------|---------------------|
| Z6187 | Zolpidem | 97310-7 |
| Z6185 | Phenobarbital | 97300-8 |
| Z6186 | Butalbital | 97224-0 |
| Z6184 | Phencyclidine | 97298-4 |
| Z6182 | Meprobamate | 97270-3 |
| Z6183 | Carisoprodol | 97226-5 |
| Z6302 | Xylazine | Not Provided |

| | | |
|-------|--------------------------------|--------------|
| Z6181 | Dextro/Levo Methorphan | 97588-8 |
| Z6179 | Delta-9 Carboxy THC | 97584-7 |
| Z6180 | Delta-9 THC | 97586-2 |
| Z6176 | Benzoylcegonine | 97222-4 |
| Z6177 | Cocaethylene | 97232-3 |
| Z6178 | Cocaine | 97234-9 |
| Z6171 | Amphetamine | 97220-8 |
| Z6172 | MDA | 97262-0 |
| Z6173 | MDEA | 97264-6 |
| Z6174 | MDMA | 97266-1 |
| Z6175 | Methamphetamine | 97274-5 |
| Z6168 | alpha-PVP | 97216-6 |
| Z6169 | Ethylone | 97246-3 |
| Z6170 | Methylone | 97276-0 |
| Z6298 | Mitragynine | Not Provided |
| Z6167 | Acetyl Fentanyl | 97214-1 |
| Z6152 | 7-Amino Clonazepam | 97212-5 |
| Z6153 | Alprazolam | 97218-2 |
| Z6154 | Chlordiazepoxide | 97228-1 |
| Z6155 | Clonazepam | 97230-7 |
| Z6156 | Desalkylflurazepam | 97238-0 |
| Z6157 | Diazepam | 97240-6 |
| Z6158 | Flunitrazepam | 97250-5 |
| Z6159 | Flurazepam | 97252-1 |
| Z6160 | Hydroxytriazolam | 97258-8 |
| Z6161 | Lorazepam | 97260-4 |
| Z6162 | Midazolam | 97278-6 |
| Z6163 | Nordiazepam | 97282-8 |
| Z6164 | Oxazepam | 97322-2 |
| Z6165 | Temazepam | 97304-0 |
| Z6166 | Triazolam | 97308-1 |
| Z6131 | 6-Monoacetylmorphine-Free | 97210-9 |
| Z6132 | Codeine-Free | 97236-4 |
| Z6133 | Dihydrocodeine/Hydrocodol-Free | 97242-2 |
| Z6134 | Hydrocodone-Free | 97254-7 |
| Z6135 | Hydromorphone-Free | 97256-2 |
| Z6136 | Morphine-Free | 97280-2 |
| Z6137 | Norhydrocodone | 97286-9 |
| Z6138 | Noroxycodone | 97290-1 |
| Z6139 | Oxycodone-Free | 97295-0 |
| Z6140 | Oxymorphone-Free | 97296-8 |
| Z6141 | EDDP | 97244-8 |

| | | |
|-------|-----------------------|--------------|
| Z6142 | Methadone | 97272-9 |
| Z6143 | Fentanyl | 97248-9 |
| Z6144 | Norfentanyl | 97284-4 |
| Z6145 | Meperidine | 97268-7 |
| Z6146 | Normeperidine | 97288-5 |
| Z6147 | Norbuprenorphine-Free | 97590-4 |
| Z6148 | Buprenorphine-Free | 97582-1 |
| Z6149 | O-Desmethyltramadol | 97292-7 |
| Z6150 | Tramadol | 97306-5 |
| Z6151 | Tapentadol | 97302-4 |
| Z6299 | Gabapentin | Not Provided |