

Overview

Method Name

Radioimmunoassay (RIA) after acid-alcohol extraction

NY State Available

Yes

Specimen

Specimen Type

Serum

Specimen Required

Specimen Type: Serum

Container/Tube: Red or SST

Specimen Volume: 0.5 mL

Collection Instructions: Draw blood in a plain, red-top tube(s). Separate within 1 hours of collection, freeze immediately. Send 0.5 mL serum frozen.

Note:

- 1. Serum gel tube is okay, but must pour off into a plastic screw cap vial and freeze.
- 2. Minimum volume does not permit for repeat analysis.

Specimen Minimum Volume

0.1 mLNOTE: Minimum volume does not allow for repeat analysis.

Reject Due To

Other	Specimen not serum
-------	--------------------

Specimen Stability Information

Specimen Type	Temperature	Time	Special Container
Serum	Frozen	200 days	

Clinical & Interpretive

Reference Values

Age	Range (ng/mL)	Mean
Prepubertal	258 – 882	570
Pubertal	273 – 892	583
Adults	333 – 967	650

Performance

PDF Report

No

Day(s) Performed

Tuesday

Report Available

11 to 17 days

Performing Laboratory Location

Esoterix Endocrinology

Fees & Codes

Fees

- Authorized users can sign in to [Test Prices](#) for detailed fee information.
- Clients without access to Test Prices can contact [Customer Service](#) 24 hours a day, seven days a week.
- Prospective clients should contact their account representative. For assistance, contact [Customer Service](#).

Test Classification

This test was developed and its performance characteristics determined by LabCorp. It has not been cleared or approved by the Food and Drug Administration.

CPT Code Information

83519

LOINC® Information

Test ID	Test Order Name	Order LOINC® Value
FIGF2	Insulin-Like Growth Factor II	2485-1

Result ID	Test Result Name	Result LOINC® Value
-----------	------------------	---------------------

Z2324	Insulin-Like Growth Factor II	2485-1
-------	-------------------------------	--------