

# **Test Definition: ADULT**

Adulterants Survey, Random, Urine

Reporting Title: Adulterants Survey, U

Performing Location: Rochester

### **Specimen Requirements:**

Container/Tube: Plastic, 60-mL urine bottle

Specimen Volume: 1.5 mL Collection Instructions:

- 1. Collect a random urine specimen.
- 2. No preservative. Additional Information:
- 1. For situations where chain of custody is required, a Chain-of-Custody Kit (T282) is available. For chain-of-custody information, see ADLTX / Adulterants Survey, Chain of Custody, Random, Urine.
- 2. Submitting less than 1.5 mL may compromise the ability to perform all necessary testing.

# **Specimen Minimum Volume:**

1 mL

#### Forms:

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	14 days	
	Ambient	72 hours	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
20606	Creatinine, U	Numeric	mg/dL	2161-8
22312	Specific Gravity	Numeric		5810-7
23509	рН	Numeric		2756-5
23511	Oxidants	Alphanumeric		58714-7
23510	Nitrites	Alphanumeric		32710-6
30914	Comment	Alphanumeric		48767-8

LOINC and CPT codes are provided by the performing laboratory.



# **Test Definition: ADULT**

Adulterants Survey, Random, Urine

Supp	lemental	Report:
------	----------	---------

No

## **CPT Code Information:**

81005

#### **Reference Values:**

Cutoff concentrations Oxidants: 200 mg/L Nitrites: 500 mg/L