

**Test Definition: EVROL** 

Everolimus, Blood

Reporting Title: Everolimus, B Performing Location: Rochester

## **Specimen Requirements:**

Container/Tube: Lavender top (EDTA)

Specimen Volume: 3 mL Collection Instructions:

- 1. Draw blood immediately before next scheduled dose.
- 2. Do not centrifuge.
- 3. Send whole blood specimen in original tube. Do not aliquot.

Additional Information: Therapeutic range applies to trough specimens collected immediately prior to a.m. dose.

# **Specimen Minimum Volume:**

1 mL

#### Forms:

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

- -Renal Diagnostics Test Request (T830)
- -Therapeutics Test Request (T831)

Specimen Type	Temperature	Time	Special Container
Whole Blood EDTA	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	14 days	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
35146	Everolimus, B	Numeric	ng/mL	50544-6

LOINC and CPT codes are provided by the performing laboratory.

### Supplemental Report:

No

#### **CPT Code Information:**

80169



# **Test Definition: EVROL**

Everolimus, Blood

### **Reference Values:**

3-8 ng/mL

Target steady-state trough concentrations vary depending on the type of transplant, concomitant immunosuppression, clinical/institutional protocols, and time post-transplant. Results should be interpreted in conjunction with this clinical information and any physical signs/symptoms of rejection/toxicity.