

Test Definition: FUROC

UroVysion for Detection of Bladder Cancer, Urine

Reporting Title: UroVysion (R) for Bladder Cancer

Performing Location: Rochester

Necessary Information:

1. Specimen source is required on request form (ie, voided, catheterized, bladder washing).

2. Provide fixative, source, reason for referral (evaluate for urothelial carcinoma or hematuria) and status of diagnosis (known previous diagnosis or suspected/unknown).

Specimen Requirements:

Specimen Type: Urine

Sources: Voided urine, catheterized urine, bladder washings, stoma collections, ureteral brushings or washings, renal

pelvic brushings or washings

Supplies: FISH for Urothelial Carcinoma Urine Collection Kit (T509)

Container/Tube:

Preferred: FISH for Urothelial Carcinoma in Urocyte Urine Collection Kit

Acceptable: 70% ethanol, PreservCyt, CytoLyt, ThinPrep UroCyte (UroCyte PreservCyt Solution)

Specimen Volume: 30 mL Collection Instructions:

1. Follow instructions included with Urocyte Urine Collection Kit.

2. If kit is not used, submit a random urine specimen with an equal volume of 70% ethanol, PreservCyt, or CytoLyt.

Specimen Minimum Volume:

See Specimen Required

Forms:

- 1. Pathology/Cytology Information (T707)
- 2. If not ordering electronically, complete, print, and send an Oncology Test Request (T729) with the specimen.

Specimen Type	Temperature	Time	Special Container
Varies	Refrigerated (preferred)		
	Ambient		



Test Definition: FUROC

UroVysion for Detection of Bladder Cancer, Urine

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
54674	Result Summary	Alphanumeric		50397-9
54675	Result	Alphanumeric		82251-0
54676	Interpretation	Alphanumeric		69047-9
54677	Reason for Referral	Alphanumeric		42349-1
54678	Specimen	Alphanumeric		31208-2
54679	Source	Alphanumeric		31208-2
54680	Released By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

88120

Reference Values:

An interpretive report will be provided.