

Reporting Title: Clozapine, S**Performing Location:** Rochester**Specimen Requirements:**

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Red top (serum gel/SST are not acceptable)

Submission Container/Tube: Plastic vial

Specimen Volume: 1 mL

Collection Instructions:

1. Collect specimen immediately before next scheduled dose.

2. Within 2 hours of collection, centrifuge and aliquot serum into a plastic vial.

Additional Information: Therapeutic range (trough level) applies to specimens collected immediately prior to next dose.

Specimen Minimum Volume:

0.6 mL

Forms:

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum Red	Refrigerated (preferred)	28 days	
	Frozen	28 days	
	Ambient	7 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
33840	Clozapine	Numeric	ng/mL	6896-5
33841	Norclozapine	Numeric	ng/mL	10992-6
33842	Clozapine+Norclozapine Total	Alphanumeric	ng/mL	12375-2

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80159

Reference Values:

Clozapine

Therapeutic range: 350-600 ng/mL

Norclozapine

Therapeutic range: Not well established

Clozapine + Norclozapine

Therapeutic range: Not well established