

Reporting Title: Fungal Culture, Vaginal **Performing Location:** Rochester

Shipping Instructions:

Specimen should arrive within 24 hours of collection.

Necessary Information:

Specimen source is required.

Specimen Requirements:

Specimen Type: Swab Source: Vaginal secretions Container/Tube: Culture transport swab (noncharcoal) Collection Instructions:

1. Before collecting specimen, wipe away any excessive amount of secretion and discharge.

2. Obtain secretions from the mucosal membrane of the vaginal vault with a sterile swab.

3. If smear and culture are requested or both a bacterial culture and fungal culture are requested, collect a second swab to maximize test sensitivity.

Specimen Minimum Volume:

See Specimen Required

Forms:

If not ordering electronically, complete, print, and send a Microbiology Test Request (T244) with the specimen.

Specimen Type	Temperature	Time
Varies	Refrigerated (preferred)	7 days
	Ambient	7 days

Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
FVAG	Q00M0025	Specimen Source	Plain Text	No



Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
FVAG	Fungal Culture, Vaginal	Alphanumeric	mL	51723-5

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

87102-Fungal culture, vaginal
87106-Id MALDI-TOF Mass Spec Yeast (if appropriate)
87107-Id MALDI-TOF Mass Spec Fungi (if appropriate)
87107-Fungal identification panel A (if appropriate)
87107-Fungal identification panel B (if appropriate)
87150-Identification rapid PCR Coccidioides (if appropriate)
87150 x 2-Identification Histoplasma/Blastomyces, PCR (if appropriate)
87153-D2 fungal sequencing identification (if appropriate)
87150-Id, Candida auris Rapid PCR (if appropriate)

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
D2F	D2 Fungal Sequencing Identification			No	No (Bill Only)
FUNA	Fungal Ident Panel A			No	No (Bill Only)
FUNB	Fungal Ident Panel B			No	No (Bill Only)
LCCI	Ident Rapid PCR Coccidioides			No	No (Bill Only)
LCHB	Id, Histoplasma/Blastomyces PCR			No	No (Bill Only)
RMALF	Id MALDI-TOF Mass Spec Fungi			No	No (Bill Only)
RMALY	Id MALDI-TOF Mass Spec Yeast			No	No (Bill Only)

Reflex Tests:



Fungal Culture, Vaginal

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
LCCA	Id, Candida auris Rapid PCR			No	No (Bill Only)

Reference Values:

Negative

If positive, yeast will be identified.