

Reporting Title: Histamine Plasma**Performing Location:** Eurofins Viracor**Specimen Requirements:**

Specimen Type: Plasma

Collection Container/Tube: Lavender top (EDTA)

Submission Container/Tube: Plastic vial

Specimen Volume: 3 mL

Collection Instructions:

Draw 3 mL blood in a lavender-top (EDTA) tube(s). Cool immediately on ice. Centrifuge at 1500 rpm for 10 minutes at 4 degrees C. The centrifugation should be performed within 20 minutes of collection. Carefully remove 1 mL of EDTA plasma from the upper part of the tube. Freeze and send frozen in a plastic vial.

Specimen Minimum Volume:

1 mL

Specimen Type	Temperature	Time	Special Container
Plasma EDTA	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
FHSPL	Histamine Plasma	Alphanumeric		2416-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

83088

Reference Values:

<1.0 ng/mL