
Reporting Title: Neuro-Onc Expanded Panel**Performing Location:** Rochester**Ordering Guidance:**

Multiple oncology (cancer) gene panels are available. For more information see Hematology, Oncology, and Hereditary Test Selection Guide.

Necessary Information:

Pathology report (final or preliminary), at minimum containing the following information, must accompany specimen for testing to be performed:

1. Patient name
2. Block number-must be on all blocks, slides, and paperwork (can be handwritten on the paperwork)
3. Tissue collection date
4. Source of the tissue

Specimen Requirements:

This assay requires at least 20% tumor nuclei.

- Preferred amount of tumor area with sufficient percent tumor nuclei: tissue 360 mm(2)
- Minimum amount of tumor area: tissue 72 mm(2)
- If ordered in conjunction with CMAPT / Chromosomal Microarray, Tumor, Formalin-Fixed Paraffin-Embedded, the preferred amount of tissue is 430 mm(2), the minimum amount is 180 mm(2).
- These amounts are cumulative over up to 15 unstained slides and must have adequate percent tumor nuclei.
- Tissue fixation: 10% neutral buffered formalin, not decalcified
- For specimen preparation guidance, see Tissue Requirement for Solid Tumor Next-Generation Sequencing. For the example in this document, 6 mm x 6 mm x 10 slides are preferred: approximate/equivalent to 360 mm(2) with the minimum acceptable of 4 mm x 4 mm x 10 slides: approximate/equivalent to 144 mm(2).

Preferred:

Specimen Type: Tissue block

Collection Instructions: Submit a formalin-fixed, paraffin-embedded tissue block with acceptable amount of tumor tissue.

Acceptable:

Specimen Type: Tissue slide

Slides: 1 stained and 15 unstained

Collection Instructions: Submit 1 slide stained with hematoxylin and eosin and 15 unstained, nonbaked slides with 5-micron thick sections of the tumor tissue.

Note: The total amount of required tumor nuclei can be obtained by scraping up to 15 slides from the same block.

Additional information:

1. If the amount of tissue available is close to the minimum required, the ordering provider may be asked to prioritize between the DNA and RNA components of the assay.
2. Unused unstained slides will not be returned.

Specimen Minimum Volume:

See Specimen Required

Forms:

If not ordering electronically, complete, print, and send an Oncology Test Request (T729) with the specimen.

Specimen Type	Temperature	Time	Special Container
Varies	Ambient (preferred)		
	Refrigerated		

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
603048	Result Summary	Alphanumeric		50397-9
603049	Result	Alphanumeric		82939-0
603050	Interpretation	Alphanumeric		69047-9
603051	Additional Information	Alphanumeric		48767-8
603052	Specimen	Alphanumeric		31208-2
603053	Source	Alphanumeric		31208-2
603054	Tissue ID	Alphanumeric		80398-1
603055	Released By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

81457

Reflex Tests:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
SLIRV	Slide Review in MG			Yes	No (Bill Only)

Reference Values:

An interpretive report will be provided.