

Reporting Title: Targeted Benzodiazepine Screen, U
Performing Location: Rochester

Specimen Requirements:

Only orderable as part of profile. For more information see:
-CSMPU / Controlled Substance Monitoring Panel, Random, Urine
-TBSU / Targeted Benzodiazepine Screen, Random, Urine

Supplies: Sarstedt 5 mL Aliquot Tube (T914)
Collection Container/Tube: Plastic urine container
Submission Container/Tube: Plastic, 5-mL tube
Specimen Volume: 1 mL
Collection Instructions:
1. Collect a random urine specimen.
2. No preservative

Specimen Minimum Volume:
0.5 mL

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	28 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
604871	Alprazolam	Alphanumeric	ng/mL	94116-1
604867	Alpha-Hydroxyalprazolam	Alphanumeric	ng/mL	19325-0
604891	Alpha-Hydroxyalprazolam Glucuronide	Alphanumeric	ng/mL	94115-3
604872	Chlordiazepoxide	Alphanumeric	ng/mL	19385-4
604889	Clobazam	Alphanumeric	ng/mL	94114-6
604890	N-Desmethyclobazam	Alphanumeric	ng/mL	94113-8
604873	Clonazepam	Alphanumeric	ng/mL	19399-5
604267	7-aminoclonazepam	Alphanumeric	ng/mL	94112-0

Result ID	Reporting Name	Type	Unit	LOINC®
604874	Diazepam	Alphanumeric	ng/mL	19443-1
604880	Nordiazepam	Alphanumeric	ng/mL	19624-6
604875	Flunitrazepam	Alphanumeric	ng/mL	19466-2
604866	7-aminoflunitrazepam	Alphanumeric	ng/mL	94111-2
604876	Flurazepam	Alphanumeric	ng/mL	19474-6
604868	2-Hydroxy Ethyl Flurazepam	Alphanumeric	ng/mL	94110-4
604877	Lorazepam	Alphanumeric	ng/mL	19520-6
604878	Lorazepam Glucuronide	Alphanumeric	ng/mL	94109-6
604879	Midazolam	Alphanumeric	ng/mL	19585-9
604869	Alpha-Hydroxy Midazolam	Alphanumeric	ng/mL	94108-8
604881	Oxazepam	Alphanumeric	ng/mL	19638-6
604882	Oxazepam Glucuronide	Alphanumeric	ng/mL	94107-0
604883	Prizepam	Alphanumeric	ng/mL	19678-2
604884	Temazepam	Alphanumeric	ng/mL	19698-0
604885	Temazepam Glucuronide	Alphanumeric	ng/mL	94106-2
604886	Triazolam	Alphanumeric	ng/mL	19714-5
604870	Alpha-Hydroxy Triazolam	Alphanumeric	ng/mL	94105-4
604887	Zolpidem	Alphanumeric	ng/mL	94104-7
604888	Zolpidem Phenyl-4-Carboxylic acid	Alphanumeric	ng/mL	94103-9
604949	Benzodiazepine Interpretation	Alphanumeric		69050-3

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80347

G0480 (if appropriate)

Reference Values:

Only orderable as part of profile. For more information see:

- CSMPU / Controlled Substance Monitoring Panel, Random, Urine
- TBSU / Targeted Benzodiazepine Screen, Random, Urine

Not Detected

Cutoff concentrations:

Alprazolam: 10 ng/mL
Alpha-hydroxyalprazolam: 10 ng/mL
Alpha-hydroxyalprazolam glucuronide: 50 ng/mL
Chlordiazepoxide: 10 ng/mL
Clobazam: 10 ng/mL
N-desmethyclobazam: 200 ng/mL
Clonazepam: 10 ng/mL
7-Aminoclonazepam: 10 ng/mL
Diazepam: 10 ng/mL
Nordiazepam: 10 ng/mL
Flunitrazepam: 10 ng/mL
7-Aminoflunitrazepam: 10 ng/mL
Flurazepam: 10 ng/mL
2-Hydroxy ethyl flurazepam: 10 ng/mL
Lorazepam: 10 ng/mL
Lorazepam glucuronide: 50 ng/mL
Midazolam: 10 ng/mL
Alpha-hydroxy midazolam: 10 ng/mL
Oxazepam: 10 ng/mL
Oxazepam glucuronide: 50 ng/mL
Prazepam: 10 ng/mL
Temazepam: 10 ng/mL
Temazepam glucuronide: 50 ng/mL
Triazolam: 10 ng/mL
Alpha-hydroxy triazolam: 10 ng/mL
Zolpidem: 10 ng/mL
Zolpidem phenyl-4-carboxylic acid: 10 ng/mL