

**Test Definition: VITAP** 

Vitamin A, Serum

**Reporting Title:** Vitamin A, S **Performing Location:** Rochester

### **Specimen Requirements:**

Only orderable as part of a profile. For more information see VITAE / Vitamin A and Vitamin E, Serum.

Patient Preparation: Fasting overnight (12-14 hours) (infants: collect specimen prior to next feeding)

Supplies: Amber Frosted Tube, 5 mL (T915)

Collection Container/Tube:

Preferred: Red top Acceptable: Serum gel

Submission Container/Tube: Amber vial

Specimen Volume: 0.5 mL

Collection Instructions: Within 24 hours of collection, centrifuge, and aliquot serum into an amber vial to protect from

light.

## **Specimen Minimum Volume:**

0.25 mL

Specimen Type	Temperature	Time	Special Container	
Serum	Refrigerated (preferred)	28 days	LIGHT PROTECTED	
	Frozen	28 days	LIGHT PROTECTED	
	Ambient	7 days	LIGHT PROTECTED	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
605124	Vitamin A	Numeric	mcg/dL	2923-1

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

No

#### **CPT Code Information:**

84590



# **Test Definition: VITAP**

Vitamin A, Serum

#### **Reference Values:**

Only orderable as part of a profile. For more information see VITAE / Vitamin A and Vitamin E, Serum.

0-6 years: 11.3-64.7 mcg/dL 7-12 years: 12.8-81.2 mcg/dL 13-17 years: 14.4-97.7 mcg/dL > or =18 years: 32.5-78.0 mcg/dL