

# **Test Definition: KIDST**

Kidney Stone Analysis

**Reporting Title:** Kidney Stone Analysis

Performing Location: Rochester

#### **Necessary Information:**

Specimen source is required.

#### **Specimen Requirements:**

Supplies: Stone Analysis Collection Kit (T550) Sources: Bladder, kidney, prostatic, renal, or urinary Specimen Volume: Entire dried calculi specimen

Collection Instructions:

- 1. Have patient collect specimen using the Patient Collection Instructions for Kidney Stones.
- 2. For detailed instructions on kidney stone preparation and packaging, see Kidney Stone Packaging Instructions.
- 3. Prepare stone by cleaning any blood or foreign material from the stone with deionized water.
- 4. Place stone on a clean filter or paper towel and let dry at ambient temperature for a minimum of 24 hours.
- 5. Do not place stone directly in a bag. If specimen is received in a bag, either transfer stone into a screw-capped, plastic container or place bag containing stone in a screw-capped, plastic container.
- Indicate source of specimen on the outside of the container (eg, left kidney, bladder, right ureter).
- 7. Repeat steps 2 through 5 for each stone received.

Specimen Stability Information: Ambient (preferred) 2 years/Refrigerated 1 year/Frozen 1 year

# **Specimen Minimum Volume:**

Entire stone

#### Forms:

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

- -General Request (T239)
- -Renal Diagnostics Test Request (T830)

Specimen Type	Temperature	Time	Special Container
Stone	Ambient (preferred)		
	Frozen	365 days	
	Refrigerated	365 days	

# **Test Definition: KIDST**

Kidney Stone Analysis

# Ask at Order Entry (AOE) Questions:

Test ID	Question ID	Description	Туре	Reportable
KIDST	SRC1	Source::  • Bladder  • Passed Stone  • Prostate  • Urethra  • Urinary Tract  • Kidney  • Left Kidney  • Right Kidney  • Renal  • Left Renal  • Right Renal  • Ureter  • Left Ureter  • Right Ureter	Answer List	Yes

### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
605761	Kidney Stone Analysis	Alphanumeric		40787-4
SRC1	Source:	Alphanumeric		31208-2
605762	Stone Interpretation	Alphanumeric		56119-1
618163	Result Comment	Alphanumeric		77202-0

LOINC and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No

# **CPT Code Information:**

82365



# **Test Definition: KIDST**

Kidney Stone Analysis

### **Reference Values:**

The presence of a kidney stone is abnormal. A quantitative report will be provided after analysis.