

Reporting Title: Chyluria Screen, Random, U**Performing Location:** Rochester**Necessary Information:**

Indicate patient's age and sex.

Specimen Requirements:

Patient Preparation: Before specimen collection, the patient should not ingest foods or beverages rich in vitamin C or take vitamin C supplements.

Supplies: Urine Container, 60 mL (T313)

Container/Tube: Plastic, 60-mL urine bottle

Specimen Volume: 15 mL

Collection Instructions: Collect a first-morning, random urine collection.

Specimen Minimum Volume:

15 mL

Specimen Type	Temperature	Time	Special Container
Urine	Frozen (preferred)	10 days	
	Refrigerated	10 days	
	Ambient	4 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
CHOLU	Cholesterol, Random, U	Numeric	mg/dL	14444-4
TRIGU	Triglycerides, Random, U	Numeric	mg/dL	14450-1
CMTCS	Interpretive Comment	Alphanumeric		95807-4

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82664-Electrophoretic technique, not elsewhere specified (Chylomicrons and lipoproteins)

84311-Spectrophotometry, analyte not specified (Cholesterol)

84478-Triglycerides

Reference Values:

No lipoproteins present