

Hemoglobin Electrophoresis Evaluation, Blood

Reporting Title: Hb Electrophoresis Evaluation

Performing Location: Rochester

Ordering Guidance:

Multiple hematology evaluations are available. For information on testing that is performed with each evaluation, see Benign Hematology Evaluation Comparison.

Necessary Information:

At minimum, include recent transfusion information and most recent complete blood cell count results.

Metabolic Hematology Patient Information (T810) is strongly recommended. Testing may proceed without this information, however if the information requested is received, any pertinent reported clinical features and data will drive the focus of the evaluation and be considered in the interpretation.

The laboratory has extensive experience in hemoglobin variant identification and many cases can be confidently classified without molecular testing. However, molecular confirmation is always available, subject to sufficient sample quantity (eg, multiplex ligation-dependent probe amplification testing requires at least 2 mL of sample in addition to protein testing requirements). If no molecular testing or specific molecular tests are desired, utilize the appropriate check boxes on the form. If the form or other communication is not received, the reviewing hematopathologist will select appropriate tests to sufficiently explain the protein findings, which may or may not include molecular testing.

Specimen Requirements:

Container/Tube:

Preferred: Lavender top (EDTA)

Acceptable: Yellow top (ACD solution B), green top (sodium heparin)

Specimen Volume: 10 mL

Collection Instructions: Send whole blood specimen in original tube. Do not aliquot.

Specimen Minimum Volume:

1 mL (this volume will limit reflex testing possibilities)

3 mL if multiplex ligation-dependent probe amplification is needed

Forms:

- 1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file. The following documents are available:
- -Informed Consent for Genetic Testing (T576)
- -Informed Consent for Genetic Testing-Spanish (T826)
- 2. Metabolic Hematology Patient Information (T810)
- 3. If not ordering electronically, complete, print, and send a Benign Hematology Test Request (T755) with the specimen

Hemoglobin Electrophoresis Evaluation, Blood

Specimen Type	Temperature	Time	Special Container
Whole Blood EDTA	Refrigerated	7 days	

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
608088	Hb Electrophoresis Interpretation	Alphanumeric		49316-3
609421	Hb Electrophoresis Interp Cancel	Alphanumeric		No LOINC Needed
41927	Hb A	Numeric	%	20572-4
	Also used by tests: HGBCE			
41928	Hb F	Numeric	%	32682-7
	Also used by tests: HGBCE			
41929	Hb A2	Numeric	%	4552-6
	Also used by tests: HGBCE			
41930	Variant 1	Alphanumeric	%	24469-9
	Also used by tests: HGBCE			
41931	Variant 2	Alphanumeric	%	24469-9
	Also used by tests: HGBCE			
41932	Variant 3	Alphanumeric	%	24469-9
	Also used by tests: HGBCE			
41933	HGBCE Interpretation	Alphanumeric		78748-1
	Also used by tests: HGBCE			
65615	HPLC Hb Variant, B	Alphanumeric		No LOINC Needed

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No



Hemoglobin Electrophoresis Evaluation, Blood

Components:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
HBELI	Hb Electrophoresis Interpretation			Yes	No
HGBCE	Hb Variant, A2 and F Quantitation,B			Yes	Yes
HPLC	HPLC Hb Variant, B			Yes	No

CPT Code Information:

83020

83021

82664 (if appropriate)

83068 (if appropriate)

83789 (if appropriate)

88184 (if appropriate)

83020-26 (if appropriate)

Reflex Tests:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
HPFH	Hb F Distribution, B			No	No
MASS	Hb Variant by Mass Spec, B			No	No
SDEX	Sickle Solubility, B			No	Yes
IEF	Isoelectric Focusing, B			No	No
UNHB	Hb Stability, B			No	No
ATHAL	Alpha-Globin Gene Analysis			No	Yes
WASQR	Alpha Globin Gene Sequencing, B			No	Yes (Order WASEQ)
WBSQR	Beta Globin Gene Sequencing, B			No	Yes (Order WBSEQ)
WBDDR	Beta Globin Cluster Locus Del/Dup,B			No	Yes (Order WBDD)
WGSQR	Gamma Globin Full Gene Sequencing			No	Yes (Order WGSEQ)
HBEL0	Hb Electrophoresis Summary Interp			No	No



Hemoglobin Electrophoresis Evaluation, Blood

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Туре	Unit	LOINC®
HPFH	8270	Hb F Distribution, B	Alphanumeric		4579-9
HPFH	2104	Interpretation	Alphanumeric		59466-3
MASS	60286	Hb Variant by Mass Spec, B	Alphanumeric		No LOINC Needed
SDEX	9180	Sickle Solubility, B	Alphanumeric		6864-3
IEF	81644	Isoelectric Focusing, B	Alphanumeric		49316-3
UNHB	9095	Hb Stability, B	Alphanumeric		4639-1
ATHAL	52834	Result Summary	Alphanumeric		50397-9
ATHAL	52835	Result	Alphanumeric		82939-0
ATHAL	52836	Interpretation	Alphanumeric		69047-9
ATHAL	54871	Additional Information	Alphanumeric		48767-8
ATHAL	52837	Specimen	Alphanumeric		31208-2
ATHAL	52838	Source	Alphanumeric		31208-2
ATHAL	52839	Method	Alphanumeric		85069-3
ATHAL	52840	Released By	Alphanumeric		18771-6
WASQR	47952	Alpha Globin Gene Sequencing Result	Alphanumeric		50397-9
WASQR	47953	Interpretation	Alphanumeric		59466-3
WBSQR	47954	Beta Globin Gene Sequencing Result	Alphanumeric		50397-9
WBSQR	47955	Interpretation	Alphanumeric		59466-3
WBDDR	48356	Beta Globin Cluster Locus Del/Dup	Alphanumeric		50397-9
WBDDR	48355	Reviewed by	Alphanumeric		18771-6
WBDDR	48357	Interpretation	Alphanumeric		59466-3
WGSQR	47950	Gamma Globin Gene Sequencing Result	Alphanumeric		50397-9
WGSQR	47951	Gamma Globin Interpretation	Alphanumeric		59466-3
HBEL0	608091	Hb Electrophoresis Summary Interp	Alphanumeric		13514-5
HBEL0	608117	Reviewed By	Alphanumeric		18771-6



Hemoglobin Electrophoresis Evaluation, Blood

Reference Values:

Hemoglobin Electrophoresis Interpretation

Definitive results and an interpretative report will be provided.

Hemoglobin Variant, A2 and F Quantitation

HEMOGLOBIN A 0-30 days: 5.9-77.2% 1-2 months: 7.9-92.4% 3-5 months: 54.7-97.1% 6-8 months: 80.0-98.0% 9-12 months: 86.2-98.0% 13-17 months: 88.8-98.0% 18-23 months: 90.4-98.0%

> or =24 months: 95.8-98.0%

HEMOGLOBIN A2

0-30 days: 0.0-2.1% 1-2 months: 0.0-2.6% 3-5 months: 1.3-3.1% > or =6 months: 2.0-3.3%

HEMOGLOBIN F

0-30 days: 22.8-92.0% 1-2 months: 7.6-89.8% 3-5 months: 1.6-42.2% 6-8 months: 0.0-16.7% 9-12 months: 0.0-10.5% 13-17 months: 0.0-7.9% 18-23 months: 0.0-6.3% > or =24 months: 0.0-0.9%

VARIANT 1 0.0

VARIANT 2

0.0

VARIANT 3

0.0