

# **Test Definition: CFSMN**

ABORATORIES Cystic Fibrosis and Spinal Muscular Atrophy Carrier Screen Panel,
Varies

Reporting Title: CF and SMA Carrier Screen Panel

Performing Location: Rochester

## **Ordering Guidance:**

This test is specifically for carrier screening purposes and is not intended for diagnostic purposes. For diagnostic testing, order CFMP / Cystic Fibrosis, CFTR Gene, Variant Panel, Varies.

If the reproductive partner is also having this test performed, call the lab for a revised risk assessment.

Targeted testing for familial variants (also called site-specific or known mutation testing) is available for all genes on this panel under FMTT / Familial Variant, Targeted Testing, Varies. Call 800-533-1710 to obtain more information about this testing option.

## **Shipping Instructions:**

Specimen preferred to arrive within 96 hours of collection.

## **Necessary Information:**

If there is a family history of cystic fibrosis (CF) or spinal muscular atrophy (SMA), the known genetic variant in the family should be supplied for best interpretation of results.

#### **Specimen Requirements:**

Specimen Type: Whole blood

Patient Preparation: A previous bone marrow transplant from an allogenic donor will interfere with testing. Call

800-533-1710 for instructions for testing patients who have received a bone marrow transplant.

Container/Tube:

Preferred: Lavender top (EDTA) or yellow top (ACD)

Acceptable: Any anticoagulant Specimen Volume: 3 mL Collection Instructions:

- 1. Invert several times to mix blood.
- 2. Send whole blood specimen in original tube. Do not aliquot.

Additional Information: To ensure minimum volume and concentration of DNA is met, the preferred volume of blood must be submitted. Testing may be canceled if DNA requirements are inadequate.

### **Specimen Minimum Volume:**

1 mL

#### Forms:

1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file. The following documents are available:

Varies

- -Informed Consent for Genetic Testing (T576)
- -Informed Consent for Genetic Testing-Spanish (T826)
- 2. Molecular Genetics: Congenital Inherited Diseases Patient Information (T521)

Specimen Type	Temperature	Time	Special Container	
Varies	Ambient (preferred)			
	Frozen			
	Refrigerated			

## **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
608350	Result Summary	Alphanumeric		50397-9
608351	Result	Alphanumeric		82939-0
608352	Interpretation	Alphanumeric		69047-9
608353	Additional Information	Alphanumeric		48767-8
608354	Method	Alphanumeric		85069-3
608355	Specimen	Alphanumeric		31208-2
608356	Source	Alphanumeric		31208-2
608357	Released By	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

## **Supplemental Report:**

No

#### **CPT Code Information:**

81220

81329

81222

81479 (if appropriate for government payers)



MAYO CLINIC
LABORATORIES Cystic Fibrosis and Spinal Muscular Atrophy Carrier Screen Panel,
Varies Varies

## **Reference Values:**

An interpretive report will be provided.