

**Reporting Title:** Mercury Occupat Exp, Random, U**Performing Location:** Rochester**Specimen Requirements:**

Patient Preparation: High concentrations of gadolinium and iodine are known to interfere with most metal tests. If either gadolinium- or iodine-containing contrast media has been administered, a specimen should not be collected for 96 hours.

Supplies: Urine Tubes, 10 mL (T068)

Collection Container/Tube: Clean, plastic urine container with no metal cap or glued insert

Submission Container/Tube: Plastic, 10-mL urine tube or clean, plastic aliquot container with no metal cap or glued insert

Specimen Volume: 3 mL

Collection Instructions:

1. Collect a random urine specimen.
2. See Metals Analysis Specimen Collection and Transport for complete instructions.

**Specimen Minimum Volume:**

1.5 mL

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	7 days	
	Frozen	7 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
608893	Mercury Occupational Exposure <b>Also used by tests: HGOU</b>	Numeric	mcg/g Cr	13465-0
CRETR	Creatinine, Random, U	Numeric	mg/dL	2161-8

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

---

**Components:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
HGOU	Mercury Occupational Exposure			Yes	No
CRETR	Creatinine, Random, U			Yes	No

**CPT Code Information:**

83825  
82570

**Reference Values:**

Biological Exposure Index (BEI): <35 mcg/g creatinine prior to shift