

Test Definition: ASUCR

Arsenic/Creatinine, Ratio, with Reflex, Random, Urine

Reporting Title: Arsenic/Creat w/Reflex, Random,U

Performing Location: Rochester

Specimen Requirements:

Patient Preparation:

-Patient should not eat seafood for a 48-hour period prior to start of collection.

-High concentrations of gadolinium and iodine are known to interfere with most metal tests. If either gadolinium- or iodine-containing contrast media has been administered, a specimen should not be collected for 96 hours.

Supplies: Urine Tubes, 10 mL (T068)

Collection Container/Tube: Clean, plastic urine container with no metal cap or glued insert

Submission Container/Tube: Plastic, 10-mL urine tube or clean, plastic aliquot container with no metal cap or glued

insert

Specimen Volume: 6 mL Collection Instructions:

1. Collect urine a random urine specimen.

2. See Metals Analysis Specimen Collection and Transport for complete instructions.

Specimen Minimum Volume:

3 mL

| Specimen Type | Temperature | Time | Special Container |
|---------------|--------------------------|----------|-------------------|
| Urine | Refrigerated (preferred) | 28 days | |
| | Frozen | 28 days | |
| | Ambient | 72 hours | |

Result Codes:

| Result ID | Reporting Name | Туре | Unit | LOINC® |
|-----------|-----------------------------|---------|----------|---------|
| 608900 | Arsenic/Creatinine Ratio, U | Numeric | mcg/g Cr | 13463-5 |
| | Also used by tests: ASCU | | | |
| 608901 | Total Arsenic Concentration | Numeric | mcg/L | 5586-3 |
| | Also used by tests: ASCU | | | |
| CRETR | Creatinine, Random, U | Numeric | mg/dL | 2161-8 |

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

Test Definition: ASUCR

Arsenic/Creatinine, Ratio, with Reflex, Random, Urine

Components:

| Test ID | Reporting Name | CPT Units | CPT Code | Always Performed | Orderable Separately |
|---------|-----------------------------|-----------|----------|---------------------|-------------------------|
| ASCU | Arsenic/Creatinine Ratio, U | | | Yes | No |
| CRETR | Creatinine, Random, U | | | Yes | No |

CPT Code Information:

82175 82570

Reflex Tests:

| Test ID | Reporting Name | CPT Units | CPT Code | Always Performed | Orderable Separately |
|---------|-------------------------------|-----------|----------|---------------------|-------------------------|
| SPAS | Arsenic Speciation, Random, U | | | No | Yes |

Result Codes for Reflex Tests:

| Test ID | Result ID | Reporting Name | Туре | Unit | LOINC® |
|---------|-----------|-----------------------------|--------------|-------|---------|
| SPAS | 607692 | Inorganic Arsenic (Toxic) | Alphanumeric | mcg/L | 12481-8 |
| SPAS | 607693 | Methylated Arsenic (Toxic) | Alphanumeric | mcg/L | 53779-5 |
| SPAS | 607694 | Organic Arsenic (Non-Toxic) | Alphanumeric | mcg/L | 53778-7 |
| SPAS | 609405 | Toxic Arsenic Concentration | Alphanumeric | mcg/L | 96250-6 |
| SPAS | 607691 | Total Arsenic Concentration | Numeric | mcg/L | 5586-3 |

Reference Values:

0-17 years: Not established

> or =18 years: <24 mcg/g creatinine