

# **Test Definition: 3FBLN**

Fibulin 3 Immunostain, Technical Component Only

Reporting Title: Fibulin 3 IHC, Tech Only

Performing Location: Rochester

### **Ordering Guidance:**

This test includes only technical performance of the stain (no pathologist interpretation is performed). If diagnostic consultation by a pathologist is required order PATHC / Pathology Consultation.

# **Shipping Instructions:**

Attach the green "Attention Pathology" address label (T498) and the pink Immunostain Technical Only label included in the kit to the outside of the transport container.

# **Specimen Requirements:**

Supplies: Immunostain Technical Only Envelope (T693)

Specimen Type: Tissue

Container/Tube: Immunostain Technical Only Envelope

Preferred: 2 Unstained positively charged glass slide (25- x 75- x 1-mm) per test ordered; sections 4-microns thick

Acceptable: Formalin-fixed, paraffin-embedded tissue block

#### Forms:

If not ordering electronically, complete, print, and send an Immunohistochemical (IHC)/In Situ Hybridization (ISH) Stains Request (T763) with the specimen.

Specimen Type	Temperature	Time	Special Container
TECHONLY	Ambient (preferred)		
	Refrigerated		

#### Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
609272	Fibulin 3 IHC, Tech Only	Alphanumeric		No LOINC Needed

LOINC and CPT codes are provided by the performing laboratory.

# **Supplemental Report:**

No



# **Test Definition: 3FBLN**

Fibulin 3 Immunostain, Technical Component Only

### **CPT Code Information:**

88342-TC, primary 88341-TC, if additional IHC

# **Reflex Tests:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
IHTOI	IHC Initial, Tech Only			No	No
IHTOA	IHC Additional, Tech Only			No	No