

Test Definition: ELASF

Pancreatic Elastase, Feces

Reporting Title: Pancreatic Elastase, F

Performing Location: Rochester

Shipping Instructions:

Preferred shipping temperature is frozen. Refrigerated or ambient specimens received more than 72 hours post collection will be rejected.

Specimen Requirements:

Supplies: Stool container, Small (Random), 4 oz (T288)

Container/Tube: Stool container

Specimen Volume: 5 g Collection Instructions:

- 1. Collect a fresh random fecal specimen, no preservatives.
- 2. If specimen is sent refrigerate, send immediately after collection.
- 3. If specimen cannot be sent immediately, freeze specimen and send frozen.

Additional Information:

- 1. Separate specimens must be submitted when multiple tests are ordered, with the exception of CALPR / Calprotectin, Feces. If only a single specimen is collected, it must be split prior to transport.
- 2. Testing cannot be added on to a previously collected specimen.

Specimen Minimum Volume:

1 g

Forms:

If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:

- -General Request (T239)
- -Gastroenterology and Hepatology Test Request (T728)

Specimen Type	Temperature	Time	Special Container	
Fecal	Frozen (preferred)	28 days		
	Ambient	72 hours		
	Refrigerated	72 hours		



Test Definition: ELASF

Pancreatic Elastase, Feces

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
ELASF	Pancreatic Elastase, F	Numeric	mcg/g	25907-7

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

82653

Reference Values:

<100 mcg/g (Severe Pancreatic Insufficiency) 100-200 mcg/g (Moderate Pancreatic Insufficiency) >200 mcg/g (Normal)

Reference values apply to all ages.