

Test Definition: 3A5Q

Cytochrome P450 3A5 Genotype, Varies

Reporting Title: CYP3A5 Genotype, V

Performing Location: Rochester

Ordering Guidance:

Testing is available as the single gene assay (this test) and as a part of a psychotropic or focused pharmacogenomics panel.

If multiple pharmacogenomic genotype testing is desired, order PGXQP / Focused Pharmacogenomics Panel, Varies.

If genotype testing for psychotropic medications is desired, order PSYQP / Psychotropic Pharmacogenomics Gene Panel, Varies.

Additional Testing Requirements:

In general, most drugs metabolized by CYP3A5 are also metabolized by CYP3A4 and usually to a greater degree than CYP3A5. For this reason, substrates of these 2 enzymes are sometimes listed together in publications and genotyping of both genes might be needed to fully understand the metabolism of these drugs and predict phenotype. If CYP3A4 genotyping is needed, order 3A4Q / Cytochrome P450 3A4 Genotype, Varies.

Specimen Requirements:

Multiple genotype tests can be performed on a single specimen after a single extraction. See Multiple Genotype Test List for a list of tests that can be ordered together.

Submit only 1 of the following specimens:

Specimen Type: Whole blood

Container/Tube: Lavender top (EDTA)

Specimen Volume: 3 mL Collection Instructions:

1. Invert several times to mix blood.

2. Send whole blood specimen in original tube. Do not aliquot. Specimen Stability Information: Ambient (preferred)/Refrigerated

Specimen Type: Saliva

Patient Preparation: Patient should not eat, drink, smoke, or chew gum 30 minutes prior to collection.

Supplies: Saliva Swab Collection Kit (T786)

Specimen Volume: One swab

Collection Instructions: Collect and send specimen per kit instructions.

Specimen Stability Information: Ambient

Specimen Type: DNA

Container/Tube: 2 mL screw top tube Specimen Volume: 100 mcL (microliters)

Collection Instructions:

1. The preferred volume is 100 mcL at a concentration of 50 ng/mcL.

2. Include concentration and volume on tube.



Test Definition: 3A5Q

Cytochrome P450 3A5 Genotype, Varies

Specimen Stability Information: Frozen (preferred) 1 year/Ambient/Refrigerated

Specimen Minimum Volume:

Blood: 0.4 mL Saliva: 1 swab

Forms:

- 1. New York Clients-Informed consent is required. Document on the request form or electronic order that a copy is on file. The following documents are available:
- -Informed Consent for Genetic Testing (T576)
- -Informed Consent for Genetic Testing-Spanish (T826)
- 2. If not ordering electronically, complete, print, and send 1 of the following forms with the specimen:
- -Neurology Specialty Testing Client Test Request (T732)
- -Therapeutics Test Request (T831)

Specimen Type	Temperature	Time	Special Container	
Varies	Varies			

Result Codes:

Result ID	Reporting Name	Туре	Unit	LOINC®
610117	CYP3A5 Genotype	Alphanumeric		81140-6
610118	CYP3A5 Phenotype	Alphanumeric		79717-5
610119	Interpretation	Alphanumeric		69047-9
610120	Additional Information	Alphanumeric		48767-8
610121	Method	Alphanumeric		85069-3
610122	Disclaimer	Alphanumeric		62364-5
610123	Reviewed by	Alphanumeric		18771-6

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

81231-CYP3A5



Test Definition: 3A5Q

Cytochrome P450 3A5 Genotype, Varies

Reference Values:

An interpretive report will be provided.