

**Reporting Title:** Targeted Stimulant Screen, U  
**Performing Location:** Rochester

**Additional Testing Requirements:**

In most cases, no additional testing is needed after the qualitative targeted stimulant test is performed if the parent drug or metabolites found are consistent with the patients prescribed medications. However, if unexpected stimulant is found, confirmatory testing can be requested at an additional charge.

**Specimen Requirements:**

Supplies: Sarstedt 5 mL Aliquot Tube (T914)  
Collection Container/Tube: Plastic urine container  
Submission Container/Tube: Plastic, 5-mL tube  
Specimen Volume: 1 mL  
Collection Instructions:  
1. Collect a random urine specimen.  
2. No preservative

**Specimen Minimum Volume:**

0.5 mL

**Forms:**

If not ordering electronically, complete, print, and send a Therapeutics Test Request (T831) with the specimen.

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	28 days	

**Ask at Order Entry (AOE) Questions:**

Test ID	Question ID	Description	Type	Reportable
LPPS	LPPS	List prescribed stimulants	Plain Text	Yes

## Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
LPPS	List prescribed stimulants	Alphanumeric		29305-0
610273	Methamphetamine <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19554-5
610274	Amphetamine <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19343-3
610275	3,4-methylenedioxymethamphetamine (MDMA) <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19568-5
610276	3,4-methylenedioxy-N-ethylamphetamine (MDEA) <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	59844-1
610277	3,4-methylenedioxyamphetamine (MDA) <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19565-1
610278	Ephedrine <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	99108-3
610279	Pseudoephedrine <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	99109-1
610280	Phentermine <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19674-1
610281	Phencyclidine (PCP) <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19659-2
610282	Methylphenidate <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	19577-6
610283	Ritalinic acid <b>Also used by tests: TSTIM</b>	Alphanumeric	ng/mL	99110-9
610284	Stimulant Interpretation <b>Also used by tests: TSTIM</b>	Alphanumeric		54247-2

LOINC and CPT codes are provided by the performing laboratory.

## Supplemental Report:

No

**Components:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
LPPS	List prescribed stimulants			Yes	No
TSTIM	Targeted Stimulant Screen, U			Yes	No

**CPT Code Information:**

80326  
G0480 (if appropriate)

**Reference Values:**

Not detected

Cutoff concentrations:  
Methamphetamine: 100 ng/mL  
Amphetamines: 100 ng/mL  
3,4-Methylenedioxymethamphetamine (MDMA): 100 ng/mL  
3,4-Methylenedioxy-N-ethylamphetamine (MDEA): 100 ng/mL  
3,4-Methylenedioxyamphetamine (MDA): 100 ng/mL  
Ephedrine: 100 ng/mL  
Pseudoephedrine: 100 ng/mL  
Phentermine: 100 ng/mL  
Phencyclidine (PCP): 20 ng/mL  
Methylphenidate: 20 ng/mL  
Ritalinic acid: 100 ng/mL