

Reporting Title: Targeted Stimulant Screen, U
Performing Location: Rochester

Specimen Requirements:

Only orderable as part of profile. For more information see:

- CSMPU / Controlled Substance Monitoring Panel, Random, Urine
- TSPU / Targeted Stimulant Screen, Random, Urine

Supplies: Sarstedt 5 mL Aliquot Tube (T914)

Collection Container/Tube: Plastic urine container

Submission Container/Tube: Plastic, 5-mL vial

Specimen Volume: 1 mL

Collection Instructions:

1. Collect a random urine specimen.
2. No preservative

Specimen Minimum Volume:

0.5 mL

Specimen Type	Temperature	Time	Special Container
Urine	Refrigerated (preferred)	14 days	
	Frozen	28 days	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
610273	Methamphetamine	Alphanumeric	ng/mL	19554-5
610274	Amphetamine	Alphanumeric	ng/mL	19343-3
610275	3,4-methylenedioxymethamphetamine (MDMA)	Alphanumeric	ng/mL	19568-5
610276	3,4-methylenedioxy-N-ethylamphetamine (MDEA)	Alphanumeric	ng/mL	59844-1
610277	3,4-methylenedioxyamphetamine (MDA)	Alphanumeric	ng/mL	19565-1
610278	Ephedrine	Alphanumeric	ng/mL	99108-3
610279	Pseudoephedrine	Alphanumeric	ng/mL	99109-1
610280	Phentermine	Alphanumeric	ng/mL	19674-1
610281	Phencyclidine (PCP)	Alphanumeric	ng/mL	19659-2

Result ID	Reporting Name	Type	Unit	LOINC®
610282	Methylphenidate	Alphanumeric	ng/mL	19577-6
610283	Ritalinic acid	Alphanumeric	ng/mL	99110-9
610284	Stimulant Interpretation	Alphanumeric		54247-2

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:

No

CPT Code Information:

80326

G0480 (if appropriate)

Reference Values:

Only orderable as part of profile. For more information see:

- CSMPU / Controlled Substance Monitoring Panel, Random, Urine
- TSPU / Targeted Stimulant Screen, Random, Urine

Not detected

Cutoff concentrations:

Methamphetamine: 100 ng/mL

Amphetamine: 100 ng/mL

3,4-Methylenedioxymethamphetamine (MDMA): 100 ng/mL

3,4-Methylenedioxy-N-ethylamphetamine (MDEA): 100 ng/mL

3,4-Methylenedioxyamphetamine (MDA): 100 ng/mL

Ephedrine: 100 ng/mL

Pseudoephedrine: 100 ng/mL

Phentermine: 100 ng/mL

Phencyclidine (PCP): 20 ng/mL

Methylphenidate: 20 ng/mL

Ritalinic acid: 100 ng/mL