

## **Test Definition: ROPAN**

Ro52 and Ro60 Antibodies, IgG, Serum

Reporting Title: Ro52 and Ro60 Antibodies, IgG, S

**Performing Location:** Rochester

### **Specimen Requirements:**

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Preferred: Serum gel Acceptable: Red top

Submission Container: Plastic vial

Specimen Volume: 1 mL

Collection Instructions: Centrifuge and aliquot serum into plastic vial

### **Specimen Minimum Volume:**

0.8 mL

Specimen Type	Temperature	Time	Special Container
Serum	Refrigerated (preferred)	14 days	
	Frozen	21 days	

#### **Result Codes:**

Result ID	Reporting Name	Туре	Unit	LOINC®
RO52	Ro52 Antibody, IgG, S	Numeric	CU	53017-0
	Also used by tests: RO52			
RO60	Ro60 Antibody, IgG, S	Numeric	CU	53019-6
	Also used by tests: RO60			

LOINC and CPT codes are provided by the performing laboratory.

### **Supplemental Report:**

No



# **Test Definition: ROPAN**

Ro52 and Ro60 Antibodies, IgG, Serum

### **Components:**

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
RO52	Ro52 Antibody, IgG, S			Yes	Yes
RO60	Ro60 Antibody, IgG, S			Yes	Yes

#### **CPT Code Information:**

86235 x 2

#### **Reference Values:**

Ro52 ANTIBODY, IgG <20 CU (negative) > or =20 CU (positive) Reference values apply to all ages

Ro60 ANTIBODY, IgG <20 CU (negative) > or =20 CU (positive) Reference values apply to all ages