

Reporting Title: Hepatitis B Past Exposure, S
Performing Location: Rochester

Necessary Information:

Date of collection is required.

Specimen Requirements:

Patient Preparation: For 24 hours before specimen collection, patient should not take multivitamins or dietary supplements (eg, hair, skin, and nail supplements) containing biotin (vitamin B7).

Supplies: Sarstedt Aliquot Tube, 5 mL (T914)

Collection Container/Tube: Serum gel

Submission Container/Tube: Plastic vial

Specimen Volume: 1.2 mL

Collection Instructions:

1. Centrifuge blood collection tube per manufacturer's instructions (eg, centrifuge and aliquot within 2 hours of collection for BD Vacutainer tubes).
2. Aliquot serum into plastic vial.

Specimen Minimum Volume:

0.9 mL

Specimen Type	Temperature	Time	Special Container
Serum SST	Frozen (preferred)	90 days	
	Refrigerated	6 days	
	Ambient	72 hours	

Result Codes:

Result ID	Reporting Name	Type	Unit	LOINC®
HBAGS	HBs Antigen Scrn, S Also used by tests: HBGSN	Alphanumeric		5196-1
HBCSN	HBc Total Ab Scrn, S Also used by tests: HBCSN	Alphanumeric		13952-7
HBASN	HBs Antibody Scrn, S Also used by tests: HBBSN	Alphanumeric		10900-9

Result ID	Reporting Name	Type	Unit	LOINC®
HBSQN	HBs Antibody, Quantitative, S Also used by tests: HBAB, HBABP, HBABE, HBBSN	Alphanumeric	mIU/mL	5193-8

LOINC and CPT codes are provided by the performing laboratory.

Supplemental Report:
No

Components:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
HBGSN	HBs Antigen Scrn, S			Yes	Yes
HBCSN	HBc Total Ab Scrn, S			Yes	Yes
HBBSN	HBs Antibody Scrn, S			Yes	Yes

CPT Code Information:

86706
86704
87340
87341 (if appropriate)
G0499 (if appropriate)

Reflex Tests:

Test ID	Reporting Name	CPT Units	CPT Code	Always Performed	Orderable Separately
HBGSC	HBs Antigen Screen Confirmation, S			No	No

Result Codes for Reflex Tests:

Test ID	Result ID	Reporting Name	Type	Unit	LOINC®
HBGSC	HBGSC	HBs Antigen Screen Confirmation, S	Alphanumeric		7905-3

Reference Values:

Negative
See Viral Hepatitis Serologic Profiles.