

**Reporting Title:** NT-Pro BNP, S**Performing Location:** Rochester**Necessary Information:**

Patient's age and sex are required.

**Specimen Requirements:**

Collection Container/Tube:

Preferred: Serum gel

Acceptable: Red top

Submission Container/Tube: Plastic vial

Specimen Volume: 0.5 mL

Collection Instructions:

1. Serum gel tubes should be centrifuged within 2 hours of collection.
2. Red-top tubes should be centrifuged, and the serum aliquoted into a plastic vial within 2 hours of collection.

**Specimen Minimum Volume:**

0.5 mL

**Forms:**

If not ordering electronically, complete, print, and send a Cardiovascular Test Request Form (T724) with the specimen.

Specimen Type	Temperature	Time	Special Container
Serum	Frozen (preferred)	365 days	
	Refrigerated	7 days	

**Result Codes:**

Result ID	Reporting Name	Type	Unit	LOINC®
PBNP1	NT-Pro BNP, S	Numeric	pg/mL	83107-3

LOINC and CPT codes are provided by the performing laboratory.

**Supplemental Report:**

No

**CPT Code Information:**

---

83880

**Reference Values:****Males**

0-2 day: 321-11,987 pg/mL  
3-11 day: 263-5918 pg/mL  
12 day-1 month: Not applicable  
2 month-1 year: 37-646 pg/mL  
2 years: 39-413 pg/mL  
3 years-6 years: 23-289 pg/mL  
7 years-14 years: < or =157 pg/mL  
15 years-18 years: < or =158 pg/mL  
19-39 years: <79 pg/mL  
40-44 years: < or =72 pg/mL  
45-54 years: < or =87 pg/mL  
55-64 years: < or =88 pg/mL  
> or =65 years: < or =540 pg/mL

**Females**

0-2 day: 321-11,987 pg/mL  
3-11 day: 263-5918 pg/mL  
12 day-1 month: Not applicable  
2 month-1 year: 37-646 pg/mL  
2 years: 39-413 pg/mL  
3 years-6 years: 23-289 pg/mL  
7 years-14 years: < or =157 pg/mL  
15 years-18 years: < or =158 pg/mL  
19-39 years: <160 pg/mL  
40-44 years: < or =162 pg/mL  
45-54 years: < or =141 pg/mL  
55-64 years: < or =226 pg/mL  
> or =65 years: < or =540 pg/mL